

## CELLBOOSTER<sup>®</sup> LIFT

### Hyaluronic Acid 18mg Rejuvenating Complex:

Non-crosslinked Hyaluronic Acid 18 mg (6 mg/ml) with Amino Acids and Vitamins.



#### DESCRIPTION

CELLBOOSTER<sup>®</sup> LIFT: Hyaluronic Acid Rejuvenating Complex is produced using patented CHAC Technology.

It consists of non-crosslinked Hyaluronic Acid with Amino acids and Vitamins. The device is designed to diminish the effects of aging by filling the soft tissue, preventing oxidative stress and providing a deep hydration. It stimulates collagen production, improves microcirculation and cell nutrition.

The device is a clear yellow, sterile, injectable solution for single use presented in a case of 6 × 3 mL vials.



## COMPOSITION

Each vial contains:

- Non-crosslinked Sodium Hyaluronate 18 mg
- Amino acids (Cysteine, Glutathione, Glycine, Lysine, Proline, Valine)
- Vitamins (Sodium Ascorbyl Phosphate, Biotin)
- Phosphate buffer pH 6.8-7.6 q.s.



## INDICATIONS:

- Moderate skin depression, loss of firmness in facial tissue and contours
- Dehydrated skin; visible dryness
- Skin laxity due to age-related changes
- Loss of skin tone and microcirculation
- Couperose, atrophic scars, post-acne, striae
- Fine lines and wrinkles

## FORMAT:

Box with 6 vials x 3 ml.

## PROGRAM:

### Intensive course:



3 treatments with 2-3 weeks interval.

### Support course:



1 treatment per 2 months.

## TREATMENT AREAS:


Epidermis or dermis of the face, neck, décollete area, back of the hands, internal face of the arms or body.

## PROTOCOL A): Full face treatment with CELLBOOSTER® LIFT

### SKIN PREPARATION

- Cleansing | • Antiseptic | • Topical anesthesia (optional)

### 1 PAPULA

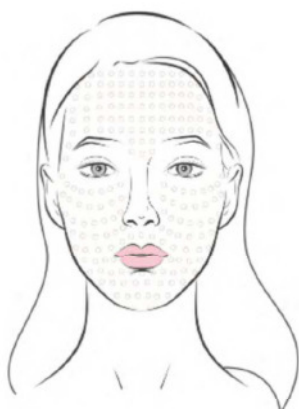

- **Injection angle:** 30°, bevel of the needle up.
- **Injection depth:** Intradermal.
- **Volume injected:** 3ml for full face, 0,025 ml per point every 2/3mm.
- **Needle type:** 30-32G, 4 mm.

### 2 FANNING



- **Injection angle:** 30°.
- **Injection depth:** hypodermis.
- **Volume injected:** 3ml for full face (1,5ml per hemiface).
- **Cannula type:** 25G or smaller.

### 3 MICROPAPULA



- **Injection angle:** 30°, bevel of the needle up.
- **Injection depth:** Intradermal, Superficial.
- **Volume injected:** 0,1-0,3 ml on each eye.
- **Needle type:** 32G or smaller, 4 mm.

### 4 RETROGRADE LINEAR

- **Injection angle:** 30°, then parallel to the skin.
- **Injection depth:** Intradermal.
- **Volume injected:** 0,02-0,05 ml on each line.
- **Needle type:** 30G or smaller, 13 mm.



### FINALIZATION

- Antiseptic
- Post injection cream

## PROTOCOL B): Decollete and neck areas treatment with CELLBOOSTER<sup>®</sup> LIFT

### SKIN PREPARATION


- Cleansing | • Antiseptic | • Topical anesthesia (optional)

### 1 FANNING / RETROGRADE LINEAR



- **Injection angle:** 30°.
- **Injection depth:** dermis and hypodermis border.
- **Volume injected:** 3ml for full decolletage.
- **Cannula type:** 25G or smaller.

### 2 PAPULA

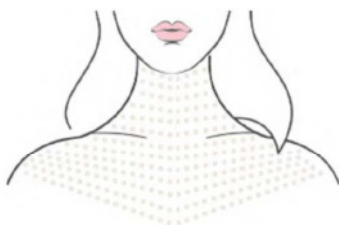

- **Injection angle:** 30°, bevel of the needle up.
- **Injection depth:** intradermal.
- **Volume injected:** 3ml for neck and decolletage, 0,025 ml per point every 2/3 mm.
- **Needle type:** 30-32G, 4 mm.


### 3 MICRO BOLUS



- **Injection angle:** 30°, bevel of the needle up.
- **Injection depth:** intradermal, space between injections 1 mm.
- **Volume injected:** 0,5-1ml for necklace lines.
- **Needle type:** 30-32G, 4 mm.

### 4 RETROGRADE LINEAR

- **Injection angle:** 30°, then parallel to the skin.
- **Injection depth:** Intradermal.
- **Volume injected:** 0,02-0,05 ml on each line.
- **Needle type:** 30G or smaller, 13 mm.

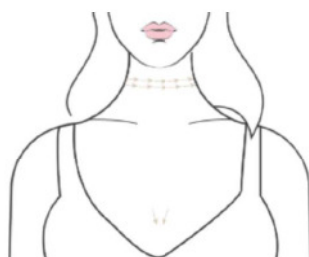

### FINALIZATION

- Antiseptic
- Post injection cream