

## CELLBOOSTER<sup>®</sup> SHAPE

### Hyaluronic Acid 18mg Reshaping Complex:

Non-crosslinked Hyaluronic Acid 18 mg (6 mg/ml) with Vitamin C and L-Carnitine.



#### DESCRIPTION

CELLBOOSTER<sup>®</sup> SHAPE: Hyaluronic Acid Reshaping Complex is produced using patented CHAC Technology.

It consists of non-crosslinked Hyaluronic Acid with L-Carnitine. The device is designed for reducing local fat accumulation or deposit. It reduces water retention and diminishes puffiness by improving microcirculation and hydration.

The device is a clear colorless, sterile, injectable solution for single use presented in a case of 6 × 3 mL vials.



**COMPOSITION**

Each vial contains:

- Non-crosslinked Sodium Hyaluronate 18 mg;
- L-Carnitine, Sodium Ascorbyl Phosphate;
- Phosphate buffer pH 6.8-7.6 q.s.



**INDICATIONS:**

- Local fat accumulation or deposit
- Cellulite
- Water retention
- Puffiness, edema
- Double chin
- Cheeks and eye bags

**FORMAT:**

Box with 6 vials x 3 ml.

**PROGRAM:**

**Intensive course:**



3 treatments with 2-3 weeks interval.

**Support course:**



1 treatment per month.

**TREATMENT AREAS:**

Face, neck, décollete area, back of the hands, internal face of the arms or body.

## PROTOCOL A): Full face treatment with CELLBOOSTER<sup>®</sup> SHAPE

### SKIN PREPARATION

- Cleansing | • Antiseptic | • Topical anesthesia (optional)

### 1 FANNING



- **Injection angle:** 30°.
- **Injection depth:** subcutaneous fat.
- **Volume injected:** 3ml for full face (1,5ml per hemiface).
- **Cannula type:** 25G or smaller.

### 2 BOLUS



- **Injection angle:** 90°.
- **Injection depth:** 6 mm - subcutaneous fat.
- **Volume injected:** 0.1ml per point, one point per square centimeter.
- **Needle type:** 30G or smaller, 13mm.


### 3 BOLUS



- **Injection angle:** 90°.
- **Injection depth:** 6 mm - subcutaneous fat.
- **Volume injected:** 0.1ml per point, one point per square centimeter.
- **Needle type:** 30G or smaller, 6 mm.

### 4 PAPULA

- **Injection angle:** 30°, bevel of the needle up.
- **Injection depth:** subcutaneous fat.
- **Volume injected:** 3ml for full face, 0,025 ml per point every 2/3mm.
- **Needle type:** 30-32G, 4 mm.

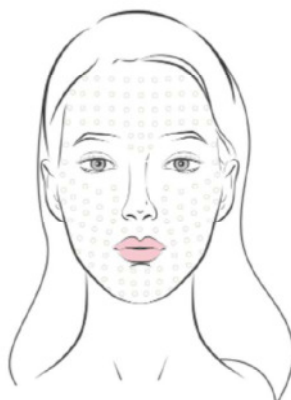

### FINALIZATION


- Antiseptic
- Post injection cream

## PROTOCOL B): Body treatment with CELLBOOSTER<sup>®</sup> SHAPE

### SKIN PREPARATION


- Cleansing | • Antiseptic | • Topical anesthesia (optional)

### 1 PAPULA

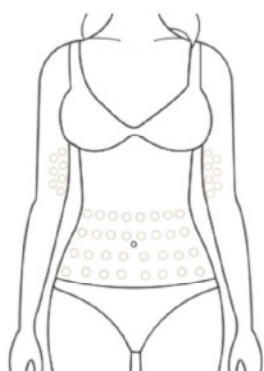


- **Injection angle:** 30°, bevel of the needle up.
- **Injection depth:** Intradermal.
- **Volume injected:** 3ml for neck and décolletage, 0,025 per point every 2/3mm.
- **Needle type:** 30-32 G, 4 mm.

### 2 BOLUS

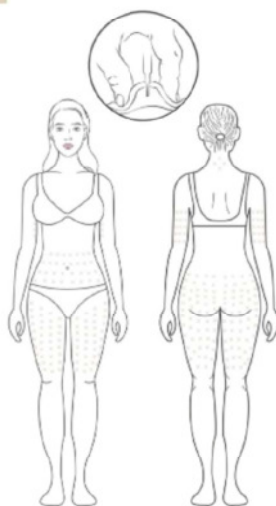

- **Injection angle:** 90°.
- **Injection depth:** 13 mm - subcutaneous fat.
- **Volume injected:** 0.1ml per point, one point per square centimeter.
- **Needle type:** 30G or smaller, 13 mm.


### 3 RETROGRADE LINEAR

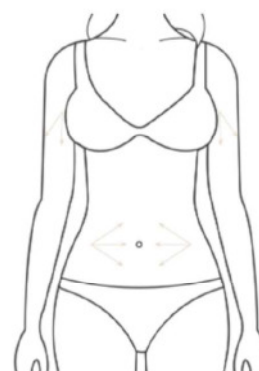

- **Injection angle:** 30°.
- **Injection depth:** subcutaneous fat.
- **Volume injected:** 3ml for one problem area.
- **Cannula type:** 25G or smaller.

### 4 RETROGRADE LINEAR

- **Injection angle:** 30°.
- **Injection depth:** subcutaneous fat.
- **Volume injected:** 3ml for one problem area.
- **Cannula type:** 25G or smaller.

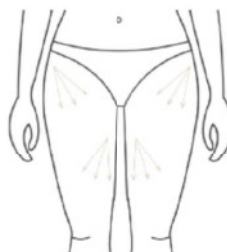

### FINALIZATION

- Antiseptic
- Post injection cream