CELLBCOSTER HAIR

Hyaluronic Acid 18mg Strengthening Complex:

Non-crosslinked Hyaluronic Acid (6mg/ml) with Amino Acids and Vitamins



DESCRIPTION

 ${\tt CELLBOOSTER} @ \ {\tt HAIR: Hyaluronic\ Acid\ Strengthening\ Complex\ is\ produced\ using\ patented\ CHAC\ Technology}.$

It consists of non-crosslinked Hyaluronic Acid with Amino Acids and Vitamins. The device is designed for strengthening hair follicles, improving hair growth and microcirculation.

It restores scalp skin structure and provides hair fiber restructuring.

The device is a clear, colorless to light yellow, sterile, injectable solution for single use presented in a case of 6×3 mL vials.

Swiss Quality •



COMPOSITION

Each vial (3ml) contains:

- Non-crosslinked Hyaluronic Acid 18 mg (6mg/ml)
- Amino Acids: Arginine, Cysteine, Glutamine, Glycine, Lysine
- Vitamins: Niacin, Pantothenic Acid (B5), Pyridoxine (B6), Biotin (B7), Cyanocobalamin (B12), Rutin
- Copper Gluconate, Zinc Carbonate.
- Phosphate buffer pH 6.8-7.6 q.s.



INDICATIONS:

- · Non cicatricial alopecia
- Alopecia areata
- Damaged hair shaft (brittle, dull color, split ends)
- · Androgenic (androgenetic) alopecia
- Premature graying
- · Seborrhea, psoriasis

FORMAT:

Box with 6 vials x 3 ml.

PROGRAM:

Intensive course:



6 treatments with 2-3 weeks interval.

Support course:



1 treatment per month

TREATMENT AREAS:

Epidermis and dermis of the scalp near hair roots according to the mesotherapy technique.

22 | Swiss Quality ■

CELLBCOSTER: HAIR

PROTOCOL A): Scalp treatment with CELLBCOSTER HAIR

SCALP PREPARATION

Antiseptic

Anesthesia (optional)

MICROPAPULA



- Injection angle: 30°, bevel of the needle up.
- Injection depth: intradermal.
- Volume injected: 3ml for scalp area, 0,025 ml per point every 2/3mm.
- Needle type: 30-32G, 4 mm.

MICROPAPULA



- Injection angle: 30°, bevel of the needle up.
- · Injection depth: intradermal.
- Volume injected: 3ml for scalp area, 0,025 ml per point every 2/3mm.
- Needle type: 30-32G, 4 mm.

3 NAPAGGE



- Injection angle: 30°, bevel of the needle up.
- Injection depth: Epidermal, dermal border.
- Volume injected: 3ml for full scalp area, 0,025 ml per point every 2/3mm.
- Needle type: 30-32G, 4 mm.

4 MICROPAPULA

- Injection angle: 30°, needle bevel of the needle up.
- Injection depth: Intradermal.
- Volume injected: 3 ml for beard area, 0,025 ml per injection every point every 2/3 mm.
- Needle type: 30-32G, 4 mm.



FINALIZATION

•Antiseptic